## Hodges Library Assistive Hearing Technology





2. 📣 Attach Mic to Speaker Transmitter.



On top of the Speaker Transmitter, press and Hold the On/Off/Mute Button for 3 seconds to turn on the Speaker Transmitter – the green light should illuminate.





4. Position the microphone as close to the speaker's mouth as possible. To adjust speaker volume, select the Menu button and use the left/right arrows to adjust the volume up or down.



5. Set up the Receiver Transmitter



6. Attach headset or personal wired earphones into the top of the Receiver Transmitter.



Rotate volume dial on top of the Receiver Transmitter from Off to a number– a green light should illuminate. This dial will also serve as your volume control.



8. Open the battery compartment. Select Channel Seek. This will connect the receiver transmitter to the speaker transmitter.





9. Adjust the volume on the receiver as needed.

